

HOW TO: CLEAN ULTRASONIC BATHS




Ultra P

NORMAL SOILAGE	
Powder	Working Solution
1 full scoop (10g)	1 Litre warm water
HEAVY SOILAGE/HARD WATER	
Powder	Working Solution
2 full scoops (20g)	1 Litre warm water



Turn off the Ultrasonic Bath and unplug, leaving to cool down.
Drain REPROZYME from the tank.

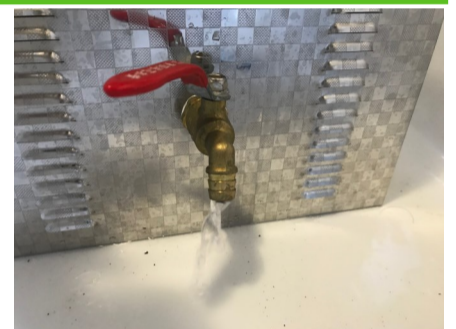

Thoroughly wipe the internal surface of bath with REPRODIS Wipes to remove any surface contaminants.
Dispose of used wipes in clinical waste.



Make a solution of Ultra P according the tank's full volume.
Fully immerse instrument tray into the solution and run on standard cycle according to manufacturer's instructions.



Empty solution and rinse instrument tray.
Wipe dry unless bath being immediately re-used.
Repeat weekly or as required dependent on level of soilage.



Bath last cleaned at ____:____ on ____/____/____ by _____ Due on: ____/____/____

For further information and for technical support call 08452 22 33 44 or visit www.medi-mark.co.uk

