

# APPLICATION INSTRUCTIONS

ANIGENE HLD<sub>4</sub>V has been specifically formulated to enable any user a simple and effective cleaning and disinfection product that is versatile and compatible with all major hard surface areas, especially modern synthetic flooring, surgical beds / tables and stainless steel animal housing.

| Application | Recommended Use | Dilution | Container Size | ml's required | Pump Depressions |
|-------------|-----------------|----------|----------------|---------------|------------------|
|-------------|-----------------|----------|----------------|---------------|------------------|

## HARD SURFACES

|   |   |  |              |         |      |     |
|---|---|--|--------------|---------|------|-----|
| Floors, Walls, Hard surfaces, work tops, Animal Runs, etc | Remove solids and debris first and then apply by use of suitable cleaning equipment. Remove excess fluid and allow to dry | GENERAL USE  | 1% (1 : 100) | 5 Litre | 50   | 2   |
|   |   | VIRUSES (Intermediate Risk / Clean conditions)     | 1% (1 : 100) | 5 Litre | 50   | 2   |
|   |   | VIRUSES (High Risk - Dirty conditions) / ISOLATION | 2% (1 : 50)  | 5 Litre | 100  | 4   |
| Examination Tables, Surgical tables and work tops         | Spray surface liberally and wipe dry with paper towel   | 2% (1 : 50)  |              | 500ml   | 10ml | n/a |
| Animal Cages / Kennels and Transport Boxes                | Spray surface liberally and wipe dry with paper towel   | 2% (1 : 50)  |              | 500ml   | 10ml | n/a |
| Bedding / Contaminated Linen                              | Add disinfectant to fabric conditioner tray and wash as per manufacturers recommendation                                  | 75ml per 5Kg load                                  |              |         | 75   | 3   |

## EQUIPMENT

|   |   |              |         |     |   |
|---|---|--------------|---------|-----|---|
| Grooming Tools, animal management devices, etc  | Remove detritus and remove hardened deposits before soaking in disinfectant for 20 minutes. Rinse before re-use | 1% (1 : 100) | 5 Litre | 50  | 2 |
| Food Bowls                                      |   |              |         |     |   |
| Litter Tray                                     |   |              |         |     |   |
| Cleaning Equipment (Mops, Brushes, Brooms, etc) | Remove debris by rinsing and soak in disinfectant solution for 20 minutes. Remove, rinse, leave to dry.         | 2% (1 : 50)  | 5 Litre | 100 | 4 |

Note: - Pump Dispensers deliver approximately 25ml per full depression

**Rinsing IS NOT required**

HLD<sub>4</sub>V is compatible with Stainless Steel 316, plastics, synthetic floors and cleaning equipment